

5. Dispose of the gauze and cap inside of your gloves in proper receptacle.
6. Re-glove and select an applicator.
7. Isolate the affected area of the tooth with cotton rolls or protect the gingival tissue of the affected tooth with petroleum jelly. Alternatively, a rubber dam can be used for isolation.
8. Clean and dry the affected tooth surface.
9. One ampule can cover up to 5 sites per patient.
10. Transfer material directly to the tooth surface with an applicator.
11. Allow to air-dry for up to 60 seconds.
12. Wipe any excess material from the tooth with a 2x2 or cotton roll.

If needed, one or two reapplications may be administered at intervals of one week.

How Supplied: 30 (0.1 mL) unit doses, 30 large applicator brushes and 30 small applicator brushes • Non-Sterile

Storage: Do not freeze or expose to extreme heat.

Caution: Federal law restricts this device to sale by or on the order of a dentist or physician.

Distributed by: Elevate Oral Care, LLC. West Palm Beach, FL 33411 • 877-866-9113 • ELE419-1116

Advantage Arrest™

Silver Diamine Fluoride 38%

Professional Tooth Desensitizer
Rx Only

Desensitizing Ingredient: Aqueous Silver Diamine Fluoride, 38.3% to 43.2% w/v

Inactive Ingredients: Purified water

Clinical Pharmacology: Product forms insoluble precipitates with calcium or phosphate in the dentinal tubules to block nerve impulses.

Indication and Usage: Treatment of dentinal hypersensitivity. For use in adults over the age of 21.

Contraindications: This product is contraindicated in patients with ulcerative gingivitis or stomatitis, or known sensitivity to silver or other heavy-metal ions. Patients with more than six affected sites, patients having had full mouth gingivectomies and patients showing abnormal skin sensitization in daily circumstances are recommended for exclusion.

Warnings: This product is intended for local application only. Not for ingestion. Protect the patient's eyes. Use caution to avoid contact with skin or clothing. In the event of exposure to eyes or skin, flush the area copiously with water and immediately seek medical consultation. This product yielded positive cytotoxicity in standard testing.

Precautions for Handling:**A. Storage Precautions**

- 1) Store in original packaging in a cool, dark place.
- 2) Use as soon as opened.

B. Advantage Arrest will stain skin, clothes, counter tops, floors and instruments brown or black. Refer to the following for stain removal:

- 1) Skin; wash immediately with water, soap, ammonia or iodine tincture and then rinse thoroughly with water. Do not use excessive methods in an attempt to remove difficult stains from skin as the stains will eventually fade.
- 2) Clothing/Countertops/Floors/Instruments; use the same procedures as with stained skin. Difficult stains may be treated with sodium hypochlorite.

Precautions for Use:

- 1) Advantage Arrest does not normally stain enamel or burnished dentin. Advise patients that soft dentin or margins of composite restorations may be stained. Staining may be reversed by gentle polishing with tincture of iodine (weak iodine solution).
- 2) Advise patients that air-drying and product application can cause momentary transient

pain to hypersensitive areas. Advantage Arrest has not been shown to cause pulpal necrosis even when soft dentin is treated.

- 3) Minimize product contact with gingiva and mucous membrane by using recommended amounts and careful application. Advantage Arrest may cause reversible short-term irritation. When applying Advantage Arrest to areas near the gingiva, apply petroleum jelly or cocoa butter and use cotton rolls to protect the gingival tissues. Alternatively, a rubber dam can be used to isolate the area.
- 4) If accidental contact occurs, thoroughly wash the area with water, saline solution or ~3% hydrogen peroxide. This includes contact with skin, clothes, floors and cabinets. Use caution to avoid transferring the material from gloved hands to other surfaces.

Adverse Reactions: Transient irritation of the gingiva has rarely been reported.

Dosage and Administration:

1. Wear proper Personal Protective Equipment when handling Advantage Arrest.
2. Place a unit-dose upright on a plastic lined tray or protected countertop.
3. Tap the unit on the counter/tray to ensure the liquid is at the bottom of the container.
4. Hold a 2x2 gauze over the ampule and snap off the cap while holding the base.